

en**REF**

32373B10

SurgiVetTM

Enviro-Pure[®]

For Veterinary Use Only



Latex Free

**Rx
ONLY**

For use by or on the order of a licensed veterinarian.

Description

The 32373B10 consists of 6 Enviro-Pure[®] canisters. Each is a 9.49 cm wide (3.74 inches), 18.29 cm high (7.2 inches) purple canister containing activated charcoal.

Function

The Enviro-Pure[®] activated charcoal canister is designed to be used where waste gas scavenging is needed within veterinary environments. It contains activated charcoal which absorbs anesthetic agents (Isoflurane, Sevoflurane, Enflurane and Desflurane) until it is saturated.

Instructions for Use with Enviro-Pure[®] canister only


1. Remove product from package and visually inspect for any damage.

WARNING! Contact Clinical Support if product or packaging is damaged.

2. Weigh the canister and note the weight.
3. Mount the canister at or below the level of the pop-off valve.
4. Attach either 19mm (3/4") or 22mm (7/8") corrugated tubing to the gas inlet port at the top of the canister.
5. Weigh canister periodically and dispose when canister has gained 100-120 grams.

NOTE! After the Enviro-Pure[®] canister becomes saturated; it is non-reusable because it retains anesthetic agents. When activated charcoal becomes saturated, anesthetic vapors may pass into the room.

6. Replace with a new canister and repeat instructions.

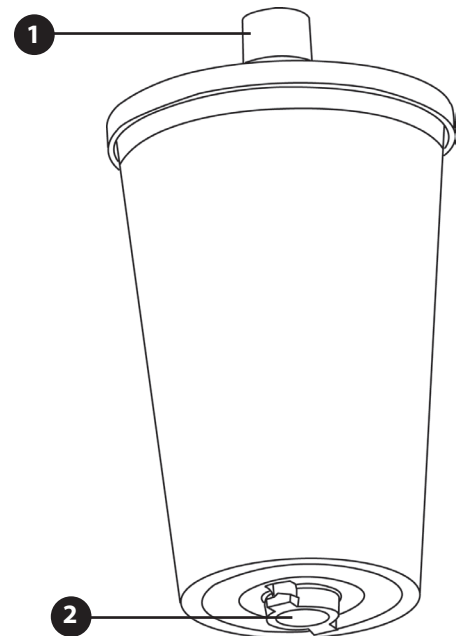

- 1 Gas Inlet Port - A 19 mm (3/4") or a 22 mm (7/8") evacuation hose is attached to this gas inlet port.
- 2 Gas Outlet Port - The gas outlet port is inserted into the (Pure-Guard[®] detector) canister socket, and then locked into place with a clockwise twist. This is done to attach the Enviro-Pure[®] charcoal canister onto the Pure-Guard[®] detector

Manufactured By: Smiths Medical PM, Inc.
 N7W22025 Johnson Drive
 Waukesha, Wisconsin 53186
 Phone: 1-262-513-8500
 Toll-Free: 1-888-745-6562 (U.S.A. only)
 Fax: 1-262-542-0718
 www.surgivet.com

SurgiVet and the Smiths design mark are trademarks of the Smiths Medical family of companies. The symbol [®] indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries.

©2009 Smiths Medical Family of companies. All rights reserved.

smiths medical

Instructions for Use with Pure-Guard® Detector

1. Remove product from package and visually inspect for any damage.

WARNING! Contact Clinical Support if product or packaging is damaged.

2. Align the two tabs located on the bottom of the Enviro-Pure® canister with the canister socket located on top of the Pure-Guard® detector.
3. Insert the canister into the canister socket.
4. Turn the canister clockwise fully, to lock the canister in place.
5. Attach either 19mm (3/4") or 22mm (7/8") corrugated tubing to the gas inlet port at the top of the canister.
6. Power the unit and verify self-test sequence.

NOTE! Refer to Pure-Guard® manual for complete instructions.

WARNING! Do not open the Enviro-Pure® canister.

WARNING! Do not refill the Enviro-Pure® canister. It must be replaced once it becomes saturated.

WARNING! The Enviro-Pure® canister will not absorb N₂O (Nitrous Oxide) gas.

WARNING! Do not reuse the Enviro-Pure® canister after contamination by biohazards. Follow local governing ordinances regarding the disposal of the charcoal canister.

WARNING! If pathogenic organisms are suspected to have contaminated the anesthesia circuit, the Enviro-Pure® canister should also be considered contaminated. Dispose of the canister according to institutional policy for biohazardous wastes.