


## SurgiVet® Pet Oxygen Masks

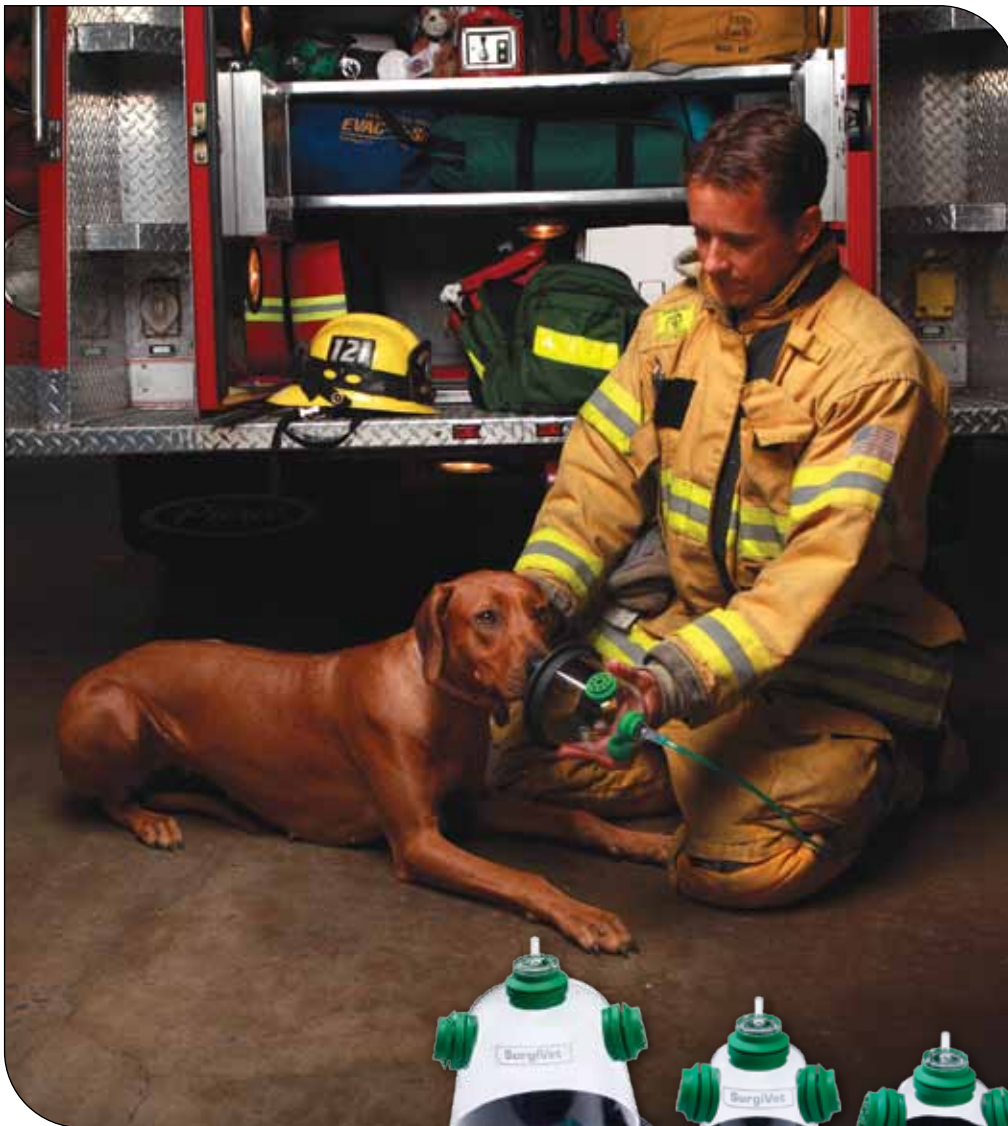

**SurgiVet® Pet Oxygen Masks** are designed to provide supplemental oxygen to animals suffering from respiratory distress, smoke inhalation, and for general recovery from anesthesia. The SurgiVet® brand of masks are used on animals by general and specialty veterinary facilities, emergency referral clinics, and fire and rescue departments.

For more information, visit [www.surgivet.com](http://www.surgivet.com).

VETERINARY

**SurgiVet**<sup>TM</sup>



V7148

## SurgiVet® Pet Oxygen Masks

- Pet Oxygen Masks can be used to provide supplemental oxygen to animals
- 22 mm and hose barb adapter included for easy connection to oxygen sources
- Dual vents to allow unrestricted inhalation and exhalation
- Each mask includes 7 feet of oxygen tubing
- 3 sizes to accommodate a variety of animals
- Latex free



## Application

1. Connect the O<sub>2</sub> source to the mask using the tubing provided or remove the 22 mm O<sub>2</sub> fitting from the mask to connect to an anesthesia machine breathing circuit to administer oxygen only.
2. Apply the mask over the patient's nose and mouth, creating a seal.
3. Set the appropriate O<sub>2</sub> flow rate for the patient:
  - Feline Mask: 1-3 liters O<sub>2</sub>
  - Small Canine Mask: 3-5 liters O<sub>2</sub>
  - Large Canine Mask: 5-7 liters O<sub>2</sub>



32490B16



32490B15



32490B10

Catalog #	Description	Size	Diaphragm Hole
32490B16	Pet Oxygen Mask Feline	3 3/4" H, 3 5/8" D	1 1/8"
32490B15	Pet Oxygen Mask Small	3 1/2" H, 3 5/8" D	1 1/4"
32490B10	Pet Oxygen Mask Large	6" H x 5 1/2" D	2 1/4"
V7148	Pet Oxygen Mask Set (One mask of each size and tubing)	See sizes above	See sizes above

SPECIFICATIONS SUBJECT TO CHANGE WITHOUT NOTICE.

Please read the Instructions for Use supplied with the product for detailed instructions, warnings and cautions.

### Smiths Medical PM, Inc.

Veterinary  
N7W22025 Johnson Drive  
Waukesha, Wisconsin 53186 USA  
Phone: 262-513-8500 Fax: 262-542-0718  
Toll-Free USA: 888-745-6562  
www.surgivet.com

### Smiths Medical Canada Ltd.

Markham, Ontario, Canada, L3R 4Y8  
Phone: 905-477-2000  
Toll-Free: 800-387-4346

Smiths Medical, part of the global technology business Smiths Group

Product(s) described may not be licensed or available for sale in Canada or other countries outside of the United States.

SurgiVet and the Smiths Medical and SurgiVet design marks are trademarks of the Smiths Medical family of companies. The symbol © indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. © 2009 Smiths Medical family of companies. All rights reserved. O<sub>2</sub> Masks Rev.00 11/09

smiths medical