

REF

HV100

SurgiVetTM

Heimlich One-Way Valve

For Veterinary Use Only

www.surgivet.com

STERILE EO

Sterilized using ethylene oxide



Latex Free



Do not reuse. Single patient, single application use.

**Rx
ONLY**

For use by or on the order of a licensed veterinarian.

Description

The Heimlich One-Way Valve consists of a valve and connecting tube ready to connect to a variety of chest drainage tubes used to treat a pneumothorax.

Function

The Heimlich One-Way Valve is intended to be used in conjunction with an appropriate chest drainage tube to treat a pneumothorax only. It is not intended to be used with any other medical condition.

Instructions for Use

1. Inspect packaging for sterility. Remove product from package and visually inspect for any damage.


WARNING! Discard if product or packaging is damaged.

2. After placement of the chest tube, attach the connecting tube (included) to the blue plastic hose barb at the end of the Heimlich One-Way Valve. The end is also labeled, "Connect chest drain to this end".
3. The opposite end with the clear hose barb will remain open, allowing the air from the chest cavity to escape and at the same time, not allowing any room air to enter.

WARNING! Do not use if fluid is present in the lungs.

WARNING! Failure to properly immobilize patient during the procedure may result in serious injury and/or death.

WARNING! Follow local governing ordinances regarding disposal of sharps and biohazardous wastes.

Manufactured By:  Smiths Medical PM, Inc.
N7W22025 Johnson Drive
Waukesha, Wisconsin 53186
Phone: 1-262-513-8500
Toll-Free: 1-888-745-6562 (U.S.A. only)
Fax: 1-262-513-9069

SurgiVet and the Smiths design mark are trademarks of the Smiths Medical family of companies. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries.

©2009 Smiths Medical Family of companies. All rights reserved.

smiths medical