

REF

CELC700, CELC850, CELC1200



Ear Lavage Catheter

For Veterinary Use Only

www.surgivet.com

STERILE EO

Sterilized using ethylene oxide



Do not reuse. Single patient, single application use.



Latex Free

Rx ONLY

For use by or on the order of a licensed veterinarian.

Description

The Ear Lavage Catheter is a single lumen open end polyurethane catheter with four distal side holes and a proximal female Luer connector. See individual package label for product specifications.

Function

The Ear Lavage Catheter is used to flush or aspirate ear wax and debris from the ear canal.

Instructions for Use

1. Inspect packaging for sterility. Remove product from package and visually inspect for any damage.

WARNING! Discard if product or packaging is damaged.

2. Sedate or anesthetize patient.
3. Restrain patient in lateral recumbency with affected ear exposed.
4. Using catheter and sterile syringe, flush ear in accordance with standard hospital protocol.
5. Dry ear canal.

WARNING! Improper use of this device may result in damage to the ear canal.

WARNING! Failure to properly immobilize patient during the procedure may result in serious injury and/or death.

WARNING! Follow local governing ordinances regarding disposal of sharps and biohazardous wastes.

Manufactured By:

Smiths Medical PM, Inc.
 N7W22025 Johnson Drive
 Waukesha, Wisconsin 53186
 Phone: 1-262-513-8500
 Toll-Free: 1-888-745-6562 (U.S.A. only)
 Fax: 1-262-513-9069



SurgiVet and the Smiths design mark are trademarks of the Smiths Medical family of companies. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries.

©2009 Smiths Medical Family of companies. All rights reserved.

smiths medical