

REF

EFC5250, EFC6250, EFC6330, EFC7250

SurgiVetTM

www.surgivet.com

Guttural Pouch Endoscopic Flushing Catheter

For Veterinary Use Only

STERILE EO

Sterilized using ethylene oxide



Latex Free



Do not reuse. Single patient, single application use.

**Rx
ONLY**

For use by or on the order of a licensed veterinarian.

Description

The Guttural Pouch Endoscopic Flushing Catheter is an open end PTFE catheter with a proximal female Luer connector and distal tip markings to aid in introduction. See individual package label for product specifications.

Function

The Guttural Pouch Endoscopic Flushing Catheter is used to lavage the lungs or guttural pouch to remove debris. The Guttural Pouch Endoscopic Catheter is used for the procedure and is removed. The catheter is not intended to be left in the patient for long term use.

Instructions for Use

1. Inspect packaging for sterility. Remove product from package and visually inspect for any damage.


WARNING! Discard if product or packaging is damaged.

2. Using an endoscope, pass the catheter into the guttural pouch.
3. Flush guttural pouch with the prescribed materials and remove catheter.

WARNING! This device is intended for one time use. It is not intended to be used as an indwelling long term use catheter.

WARNING! Failure to properly immobilize patient during the procedure may result in serious injury and/or death.

WARNING! Follow local governing ordinances regarding disposal of sharps and biohazardous wastes.

Manufactured By:  Smiths Medical PM, Inc.
N7W22025 Johnson Drive
Waukesha, Wisconsin 53186
Phone: 1-262-513-8500
Toll-Free: 1-888-745-6562 (U.S.A. only)
Fax: 1-262-513-9069

SurgiVet and the Smiths design mark are trademarks of the Smiths Medical family of companies. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries.

©2009 Smiths Medical Family of companies. All rights reserved.

smiths medical