

REF LACTEC1

SurgiVetTM


Wide Bore Infusion Set

For Veterinary Use Only

www.surgivet.com

STERILE EO Sterilized using ethylene oxide

LATEX
Latex Free

 Do not reuse. Single patient, single application use.

Rx ONLY For use by or on the order of a licensed veterinarian.

Description

The Wide Bore Infusion Set consists of a single, wide bore tubing (no chamber) with spike on one end and male Luer Lock on opposite end. See individual package label for product specifications.

Function

The Wide Bore Infusion Set is used for rapid fluid administration.


Instructions for Use

1. Inspect packaging for sterility. Remove product from package and visually inspect for any damage.

WARNING! Discard if product or packaging is damaged.

2. Remove plug from prescribed bag of fluids that are to be administered.
3. Take the Wide Bore Infusion Set and insert the end with the spike into the bag and attach the male Luer Lock into the female Luer opening of the catheter, extension set, or other appropriate fluid administration device.

WARNING! Follow local governing ordinances regarding disposal of sharps and biohazardous wastes.

Manufactured By:  Smiths Medical PM, Inc.
N7W22025 Johnson Drive
Waukesha, Wisconsin 53186
Phone: 1-262-513-8500
Toll-Free: 1-888-745-6562 (U.S.A. only)
Fax: 1-262-542-0718

SurgiVet and the Smiths design mark are trademarks of the Smiths Medical family of companies. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries.

©2009 Smiths Medical Family of companies. All rights reserved.

smiths medical