

REF

DESC1460, LACTEC3

SurgiVetTM

Bag to Bag Connector

For Veterinary Use Only

www.surgivet.com

STERILE EO

Sterilized using ethylene oxide



Latex Free



Do not reuse. Single patient, single application use.

**Rx
ONLY**

For use by or on the order of a licensed veterinarian.

Description

The Bag to Bag Connector is a double-ended spiked connecting tube with two non-vented spikes and a G-clamp. See individual package label for product specifications.

Function

The Bag to Bag Connector allows an additional bag of fluid to be connected to an existing fluid bag and delivery line when greater volumes of fluid are required and changing the bag is not possible.

Instructions for Use

1. Inspect packaging for sterility. Remove product from package and visually inspect for any damage.


WARNING! Discard if product or packaging is damaged.

2. Remove bottom plugs from both prescribed bags of fluids.
3. Insert one end of Bag to Bag Connector into the bottom of the first (ancillary) fluid bag.
4. Insert the other end of the Bag to Bag Connector into the front standard puncture area (medication/injection port) of the second (primary) fluid bag.

WARNING! The second fluid (primary) bag should be hung slightly lower than the first (ancillary) bag and the G-clamp should be left open so that the fluid will flow freely from the first (ancillary) bag to the second (primary) bag when required.

5. Insert an appropriate fluid administration device into the bottom port of the second (primary) bag to attach to the patient.

WARNING! Follow local governing ordinances regarding the disposal of sharps and biohazardous wastes.

Manufactured By:  Smiths Medical PM, Inc.
N7W22025 Johnson Drive
Waukesha, Wisconsin 53186
Phone: 1-262-513-8500
Toll-Free: 1-888-745-6562 (U.S.A. only)
Fax: 1-262-542-0718

SurgiVet and the Smiths design mark are trademarks of the Smiths Medical family of companies. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries.

©2009 Smiths Medical Family of companies. All rights reserved.

smiths medical