

en**REF**

GTIS1000, GTIS2000

SurgiVet™

Gastrostomy Tube Introduction Set

Reusable

For Veterinary Use Only

STERILE EO

Sterilized using ethylene oxide

Rx ONLY

For use by or on the order of a licensed veterinarian.



Latex Free

Description

The Gastrostomy Tube Introduction Set consists of a stainless steel threaded wire, an introduction tube, tapered catheter fixation adapter, and a percutaneous needle. See individual package label for product specifications.

Function

The Gastrostomy Tube Introduction Set is a reusable introduction set that enables non-endoscopic placement of gastrostomy tubes into the stomach. The set is designed to accept 16 or 20Fr Peg Feeding Tubes. The GTIS1000 is recommended for use with patients up to 10kg (22lbs). The GTIS2000 is recommended for use with patients over 10kg (22lbs).

Instructions for Use

1. Inspect packaging for sterility. Remove product from package and visually inspect for any damage.

WARNING! Discard if product or packaging is damaged.

2. Anesthetize patient and place patient in right lateral recumbency. Surgically prep the left lateral abdomen. Orally introduce Introduction Tube into stomach. See picture (A).
3. Use a syringe at the end of the Introduction Tube to Insufflate air to confirm position. See picture (B).
4. Insert needle percutaneously into the distal end of the Introduction Tube. See picture (C).
5. Introduce Stainless Steel Threaded Wire through the needle into the Introduction Tube and out of the mouth. See picture (D).



Picture A



Picture B



Picture C



Picture D

Manufactured By: Smiths Medical PM, Inc.
 N7W22025 Johnson Drive
 Waukesha, Wisconsin 53186
 Phone: 1-262-513-8500
 Toll-Free: 1-888-745-6562 (U.S.A. only)
 Fax: 1-262-542-0718
 www.surgivet.com

SurgiVet and the Smiths design mark are trademarks of the Smiths Medical family of companies. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries.

©2009 Smiths Medical Family of companies. All rights reserved.

smiths medical

6. Remove Introduction Tube and needle, leaving Threaded Wire. See picture (E).
7. Using a scissors, remove the tapered end of the Peg tube and insert the hose barb end of the Catheter Fixation Adapter into the PEG tube. Pass a Suture through the PEG Tube and into the hole in the Catheter Fixation Adapter and tie it. See picture (F).
8. Screw Catheter Fixation Adapter onto threaded wire. See picture (G).
9. Gently pull the PEG tube through tract and out the flank using the Threaded Wire. See picture (H).
10. Secure tube at skin using an FD100 as shown, or per standard hospital protocol. See picture (I).



Picture E

Reference: Critical Care Techniques, CD Rom, Smiths Medical PM, Inc., Waukesha, Wisconsin, USA.


WARNING! Discard if product or packaging is damaged.

WARNING! Failure to properly immobilize patient during the procedure may result in serious injury and/or death.


WARNING! Failure to place Elizabethan collar may result in dislodgement of the catheter or serious injury to the patient.

WARNING! Follow local governing ordinances regarding disposal of sharps and biohazardous wastes.


WARNING! Contraindications include, but are not limited to, pancreatitis, vomiting, or intestinal obstruction.

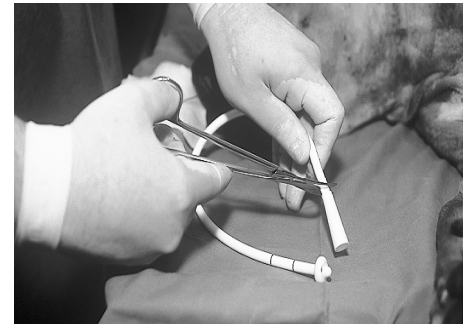

Picture F

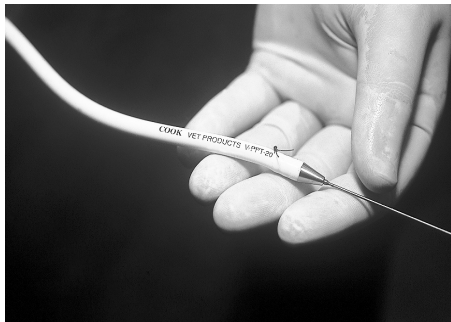

Picture G

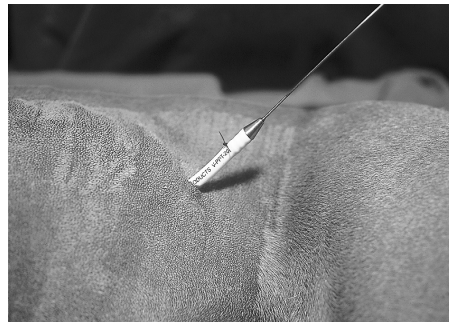

Picture H



Picture I